

# How To Print Referral PDFs

---

## Plus Optix Camera

### Printing PDFs:

1. Remove SD Card from Camera
2. Put SD Card into computer card reader
3. Open SD Card to view contents
4. Open "PDF" folder to view reports
5. Open report of child referred
6. Print Vision Screening Result

### Documenting Referral Indicators:

1. Get Screening Follow-Up Report
2. Enter Date, Screening Site, Type, # Kids Present and # Kids Screened
  - a. This information is also on the Screening Team Site Report
3. For each referral:
  - a. Enter age
  - b. Enter sex
  - c. Enter Indicator(s) as found on printed PDF Vision Screening Result
4. Follow up with each referred child's parent or guardian
5. Enter the result as reported by parent or guardian
6. Send Screening Follow-Up Report to Lion Lori Klein or PDG Rick Wagener